



Use of PC keyboard or panel (patients keyboard)

In order to make the use of RehaCom by patients as easy and convenient as possible the use of RehaCom Panel (special patients keyboard) is recommended. The RehaCom therapy software is optimized to be operated by the panel. Two large “OK” buttons are used to confirm all patient’s decisions.

If no panel (patients keyboard) is available, the therapist has to instruct the patient how to use RehaCom with the PC keyboard.

Keyboard mapping:

Panel (patients keyboard)	PC keyboard
“OK” buttons	left = space bar ; right = return ; mouse click
“Arrow” buttons	cursor keys
red “Minus” buttons	minus-key in the numeric keypad
yellow “Plus” buttons	plus-key in the numeric keypad
number “0..9” buttons	number keys

In order to make the orientation easier for the patient, a sticker set is available. Please stick the signs to the corresponding keys referring to the graphic below:

